Otolift Modul-air

User manual NL/DE/EN/FR/IT/ES

Otolift Stairlifts
Postbus 64
2860 AB Bergambacht
The Netherlands

E-mail: info@otolift.com
Internet: www.otolift.com

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General description

The Otolift Modul-air is designed for curved staircases, to move one person at the rated load comfortably throughout the journey over the staircase. Drive is by means of a rack and pinion, driven by an electric motor and powered with rechargeable batteries. The batteries are charged at the endpoints rail or optional extra charge points. Charging is done with an integrated charger in the lift. The charger is powered through the charge points, with an transformer that is connected to an electrical outlet.

The lift can be controlled in two places, an ergonomic control knob on the armrest and the remote controls. The remote controls will be mounted inline of side of the lift at both endpoints of the rail. An On/Off switch in the armrest ensures that a person can get on or off the lift safely at the endpoints.

The lift is also equipped with a rotating seat and seatbelt. The seat always rotates away from the downside of the staircase, so that the user can safely exit or enter the lift at the top of the staircase. If the optional electrical rotating seat is installed then the seat can rotate 83° to the left and right for easy exiting/boarding. On narrow staircases it may be necessary to drive rotated to avoid obstacles for the head /knees or the footplate.

Safety surfaces ensure that the lift stops when something is obstructing the lift in the direction of travel, this ensures that the lift can always be moved in the opposite direction to remove the obstruction.

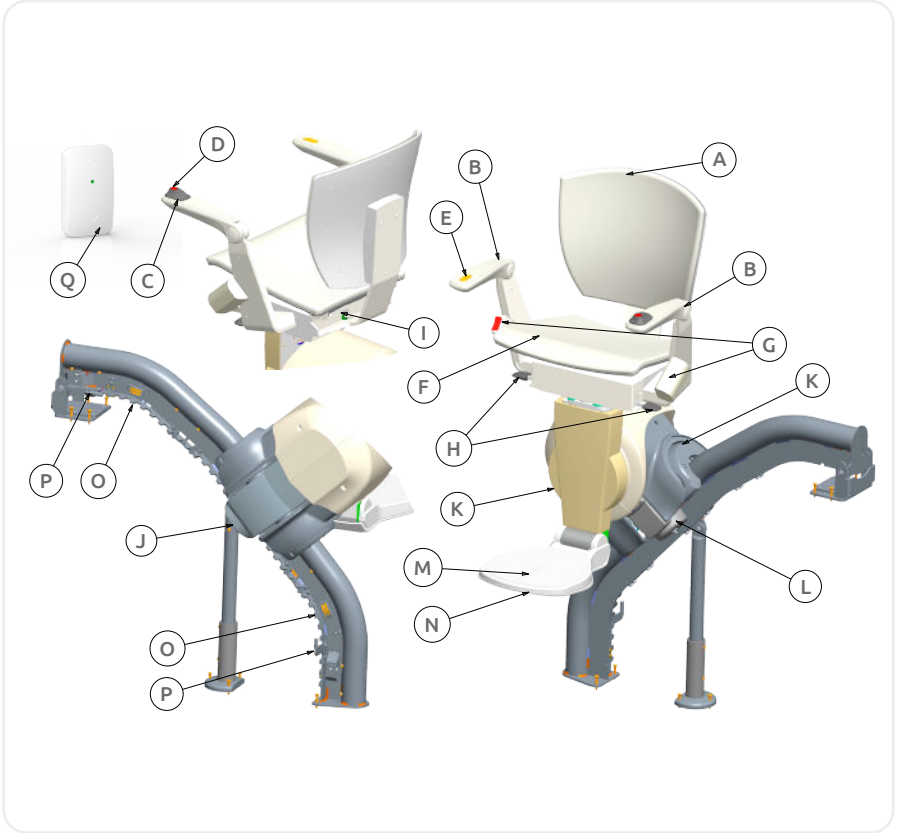
Warning/identification on the lift

Stickers on the Otolift Modul-air may never be removed or covered by any means. If the stickers have become unreadable or damaged, they must be replaced.

Improper use

- The lift should never be used by under-aged or mentally disadvantaged people without supervision.
- The lift should never be used to transport animals or goods.
- The lift should only be used as described in this user manual.
- The lift should only be used to transport one person over the complete journey.
- The lift should never be used to transport a person heavier than the rated load.
- The lift should not be used without using the seatbelt(G).
- The Otolift Modul-Air may only be installed or dismantled by Otolift trained personal.

Overview

Description		Description	
A	Backrest	J	Pressure sensitive cover/ end switches
B	Armrest	K	Carriage cover / Pressure sensitive cover
C	Control knob (up/down)	L	OSG²
D	On/off switch (stop button, red LED)	M	Footplate
E	Footplate control button (up/down)	N	Pressure sensitive edge (foot plate)
F	Seat (foldable)	O	Chargepoints³
G	Safety belt	P	End stops
H	Swivel release (manual rotating seat)	Q	Remote control (RF 433 MHz)
I	Emergency swivel release¹		

1. Electric rotating seat is an option.
2. May only be released(reset) by trained personal when activated.
3. Two charge point are standard, optional 5 extra charge points can be installed (total max. 7 charge points)

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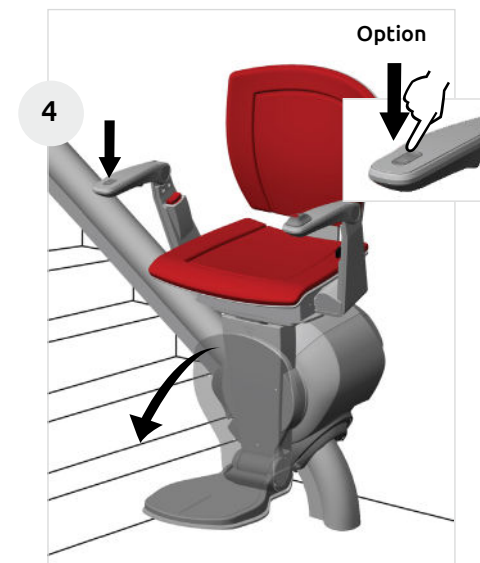
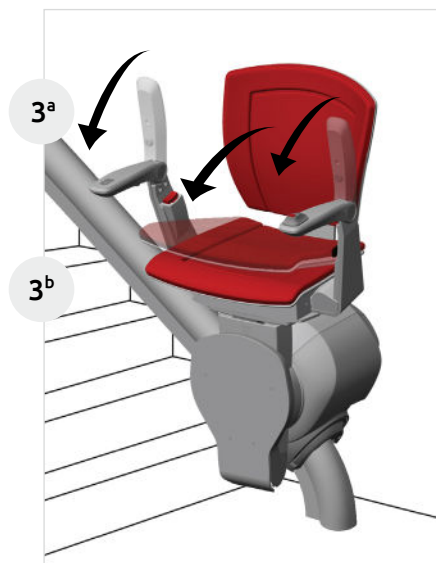
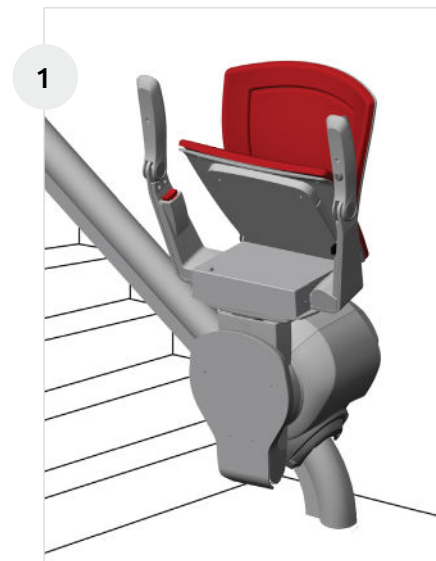
- EN**
- 1 The lift has been folded up.
 - 2 a/b Turn on the lift with the wireless key-switch.
 - 3 a/b Fold the armrests and the seat down.
 - 4 Fold the footrest down. Use the switch in the armrest to control the electrically folding footrest.

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EN

- 5 a/b Switch the lift off by pressing the control button on the armrest. Once the lift is off, the red light in the white switch will go out.
6 Secure the safety belt and check that it is locked in place.
7 a/b Switch the lift on again by pressing the control button.
8 a/b Only use the lift if the stairs are clear.

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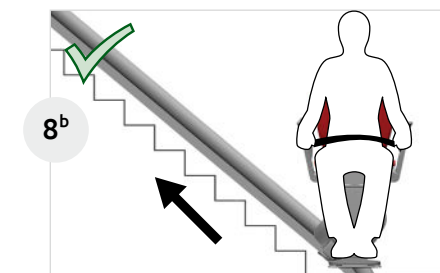
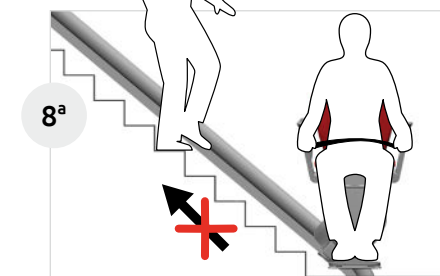
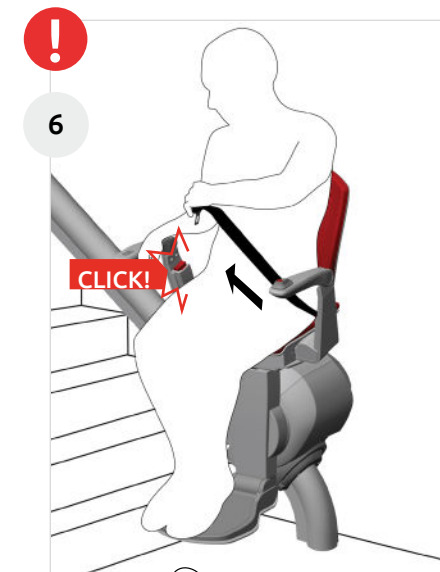
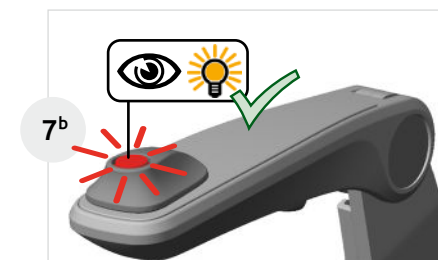
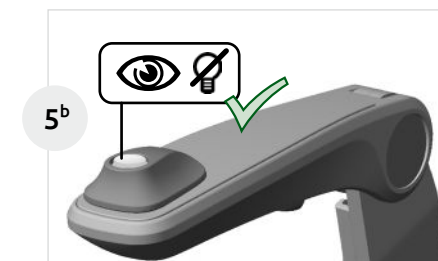
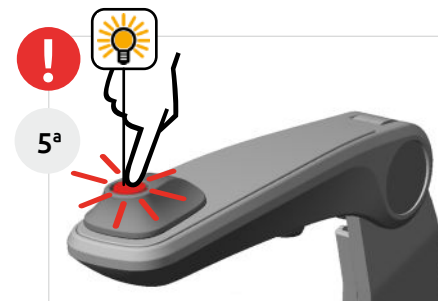
EN

- 9 Move the controls to the right or left depending on the direction in which you wish to travel.
The lift emits a beeping sound as it departs.
- 10 a The light in the armrest flashes when the lift is moving.
- 10 b Another beep is emitted once the lift has reached its destination.
- 10 c Always use the rotating seat at the top of the stairs for getting on and off the lift.

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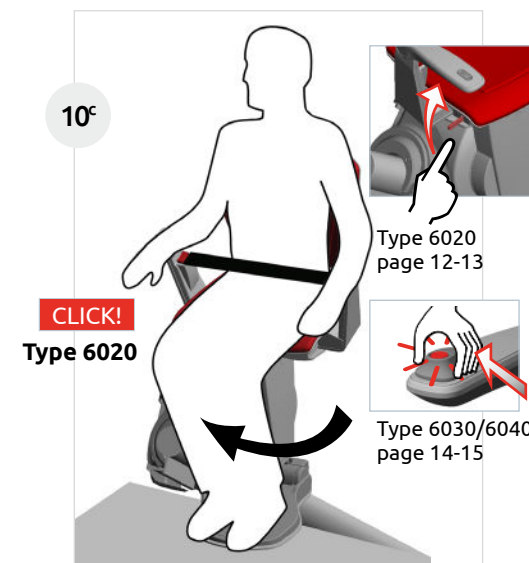
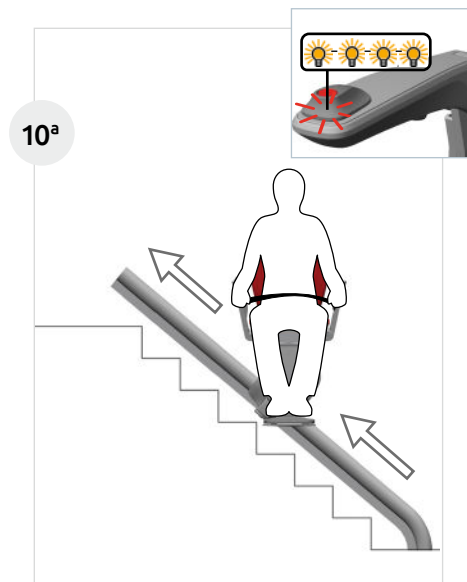
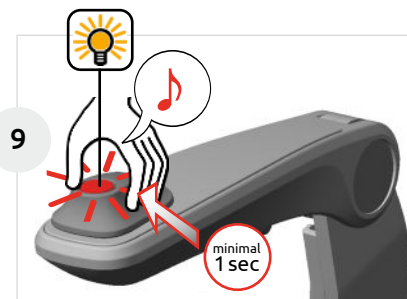
EN

- 11 a/b Switch the lift off by pressing the control button on the armrest, before you release the safety belt.
 11 c Unlock the belt by depressing the red button.
 11 d/e Turn the seat back and fold up the seat and armrests. Fold up the footrest, using the switch in the armrest if you are using the version with electronic, folding footrest.

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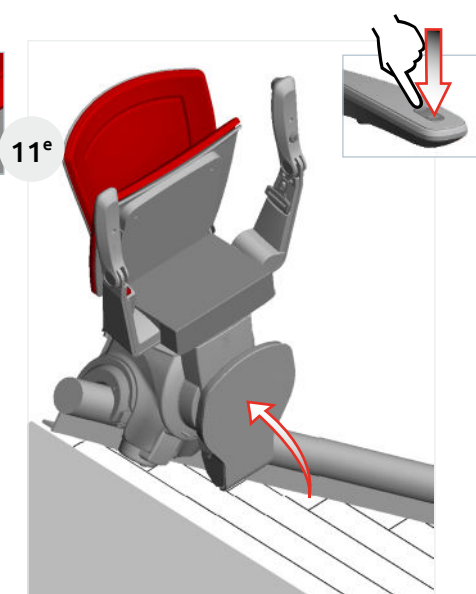
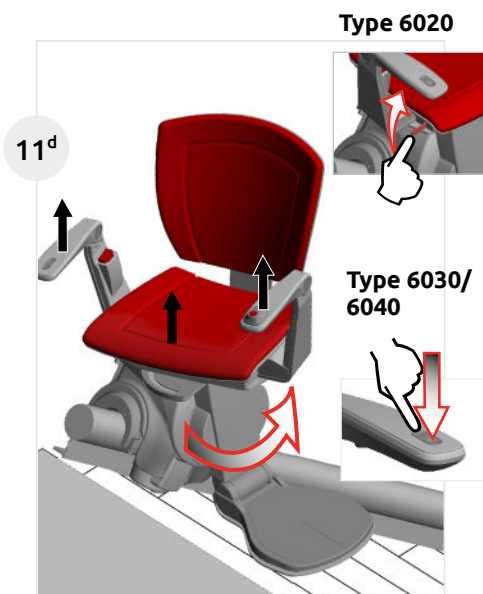
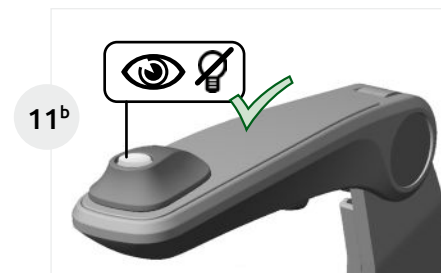
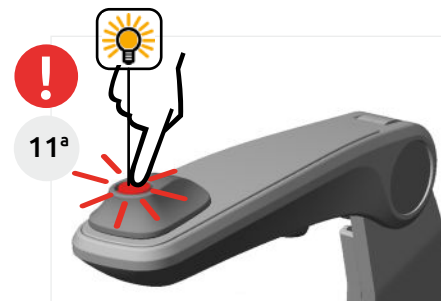
EN

Operating swivel seat model 6020:

- 12 a Fold out the Footrest, using the switch in the armrest if you are using the version with electronic, folding footrest.
- 12 b/c The swivel seat comes with levers on either side of the seat. By pulling either lever up, you can turn the seat around to face the landing, where it will lock in place when you release the lever.
- 12 d Fold up the footrest, using the switch in the armrest if you are using the version with electronic, folding footrest.

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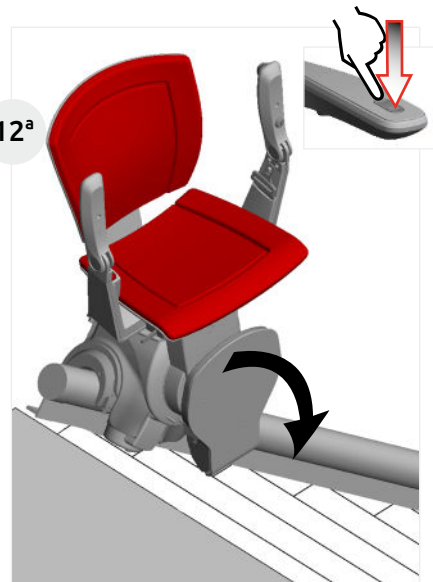
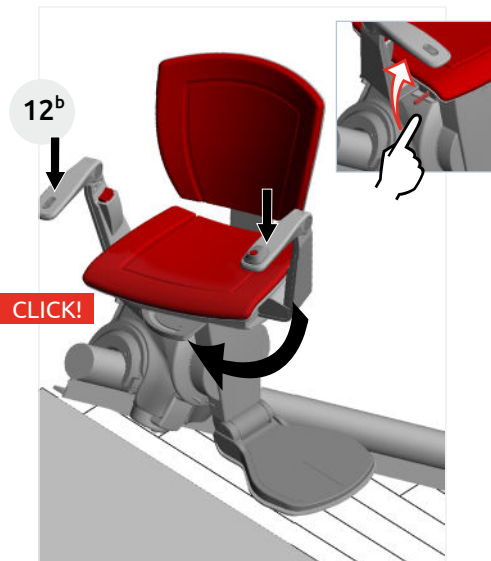
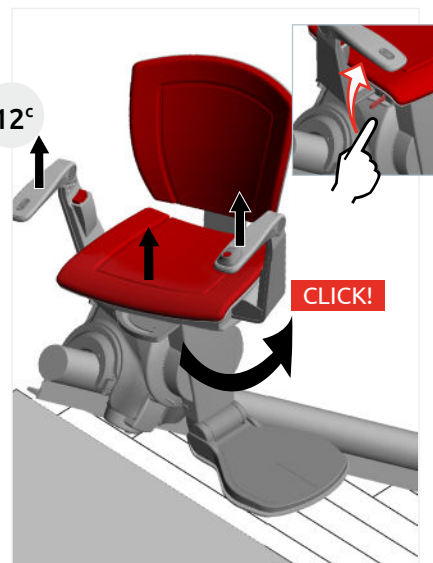
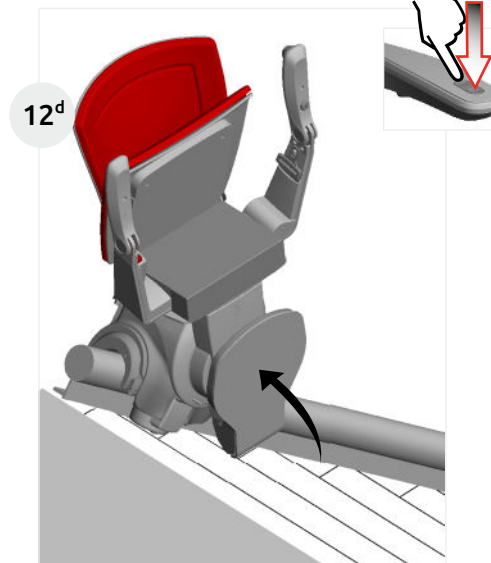
EN

Operating swivel seat model 6030/6040:

- 13 a When the stairlift is operated via the wall control and the seat is in folded position, the swivel seat drive is switched off.
- 13 b When you press the wall and footrest control button, the seat turns as the footrest unfolds.
- 13 c/d When you press the footrest control button, the seat on models 6030 and 6040 returns to its original position as the footrest folds up.

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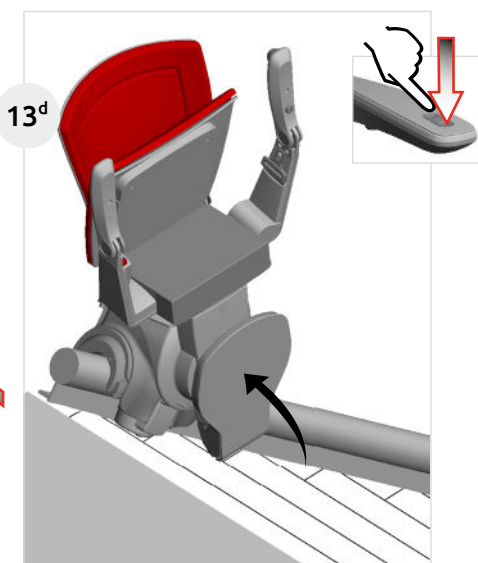
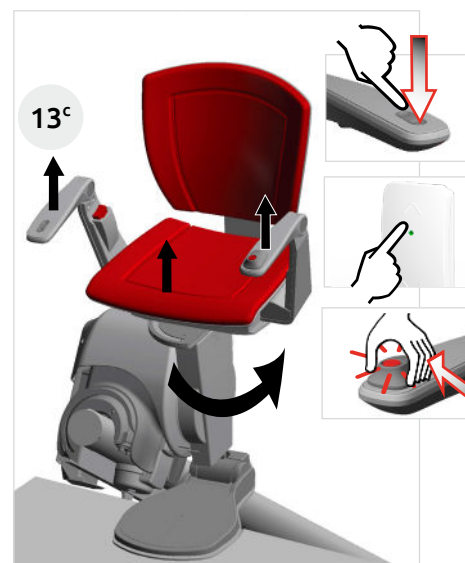
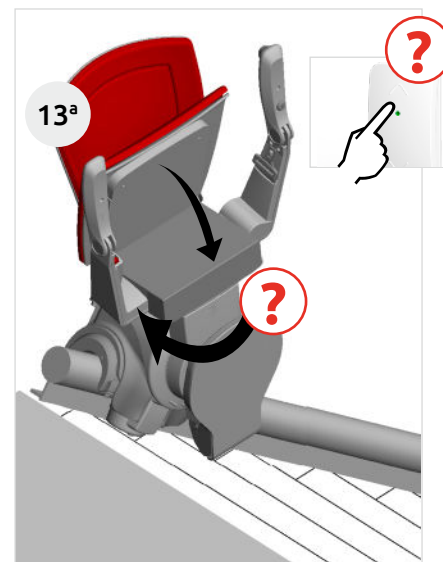
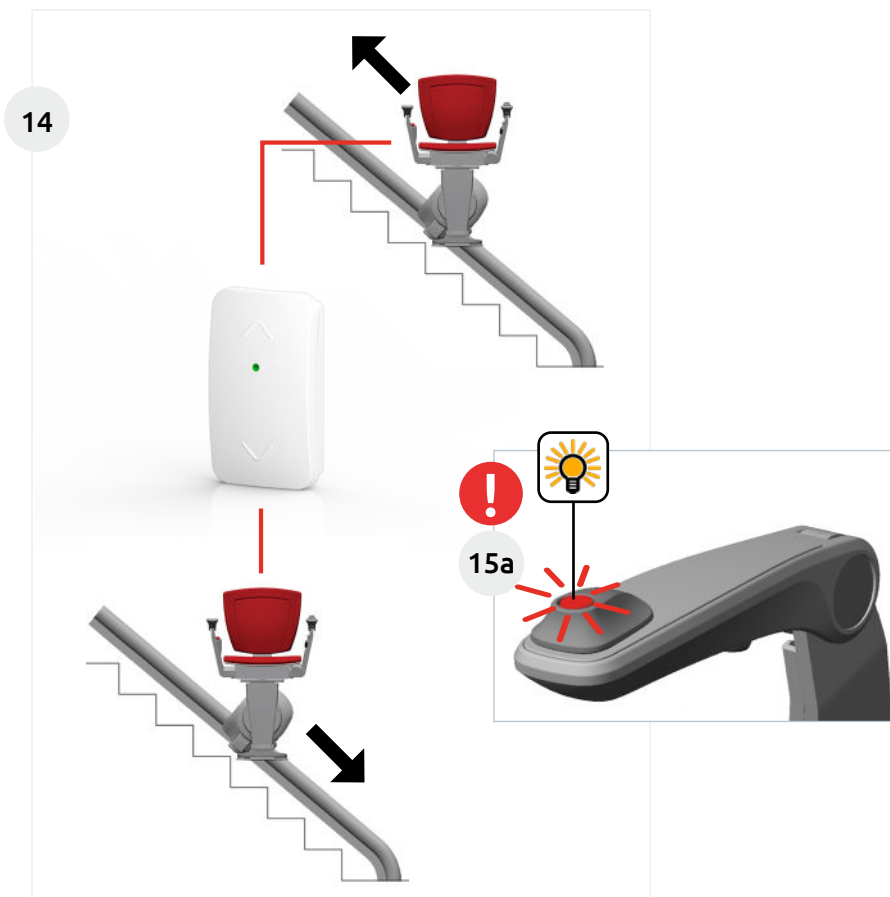
- EN** 14 With the remote control, you can call and send the lift
15 a Check if the lift is turned on. The light in the armrest must be on.
16 a With the optional wireless key-switch, you can switch the lift on and off. The lift beeps once when you lock it with the wireless key-switch. The lift beeps twice when you unlock it. If the lift is locked, the light in the armrest is switched off.

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EN

- 17 Always park the lift at a charge point (P).
The standard lift is equipped with a charge point at the upper and lower stop position. If the lift is equipped with one or more extra stop positions with a charge point, the lift will stop at one of these charge points if it is being operated via the wall control.

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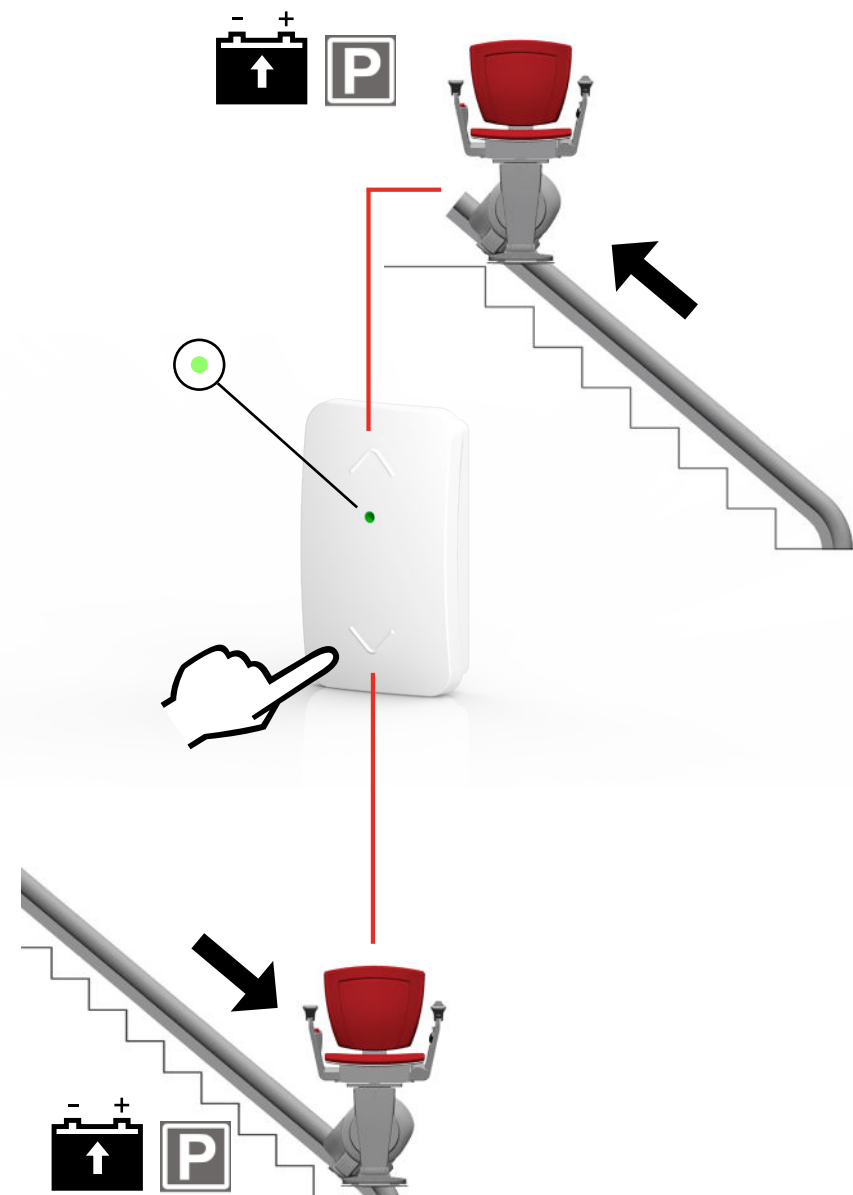
EN

Information about use during fire and the sound levels of the lift:

- 1 The stair-lift may not be used for fire-prevention purposes or evacuation.
- 2 The noise level of the stair-lift for the user is less than 70dB(A).

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EN

The lift does not work; the light in the controls flashes several times

- 1 Not illuminated: the controls are switched off by means of the wireless key-switch (see page 28).
- 2 Flashes once after releasing controls: the lift is switched off by means of the control button in the armrest.
- 3 Flashes twice after releasing controls: the rotating seat is not locked properly or is not in the correct position.

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EN

Emergency operation electrical swivel seat, get off halfway up to walk further up the stairs:

Type Modul-Air with electrical swivel seat

- 1 Lift stops halfway up due to power outage.
- 2 Unlock the seat with the lever on the back.
- 3 Hold the lever up and turn the seat toward the stairs as far as possible before releasing the lever; then check whether the seat is locked again so that you can safely get off.

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EN

Remote control does not work / key switch does not work

- 1 Check if the lift is turned on.
- 2 If the yellow LED light flashes on the remote control during use, the battery must be replaced.
- 3 Open the remote control by removing the cap. Replace the battery (penlite AA).
- 4 Is the wireless key-switch no longer working? Open the switch by removing the cap.
- 5 Remove the old battery by sliding it sideways. Install the new battery (CR2032).

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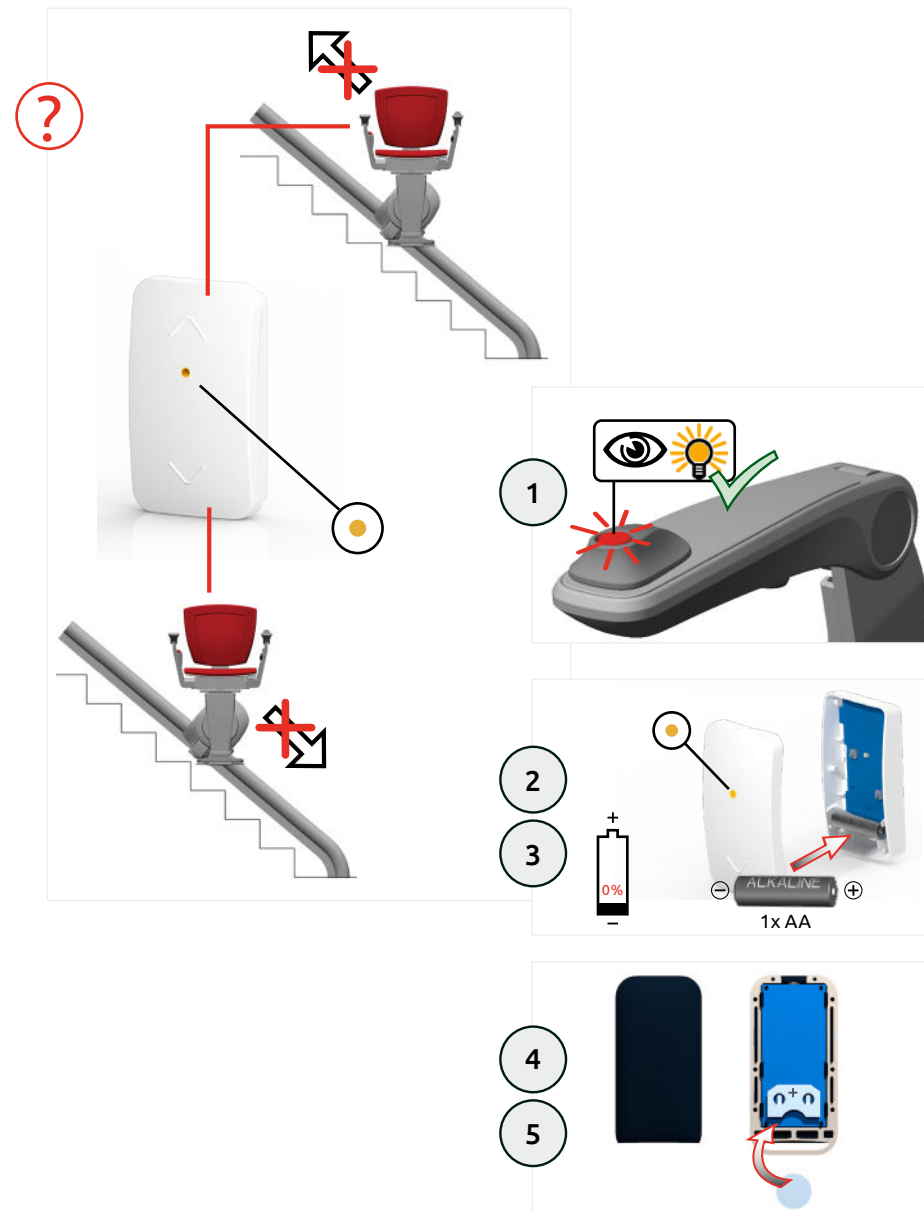
**Service Specifications
Recycling**

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EN**Care:**

Clean the stairlift regularly.

Use a mild cleaner and a soft brush or cloth to clean the seat. Do not use chlorine-based, silicone based or aggressive cleaners. Use a brush to clean the teeth of the rail and a dry cloth with alcohol based cleaning solvent to keep the rail tubes clean. Avoid excessive use of water.

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EN**Periodic maintenance and service:**

The stairlift requires periodic preventive maintenance to ensure long-term, trouble-free operation. It is recommended that your stairlift be checked and serviced every 12 months by your supplying Dealer. The following components need to be checked at least once every two years: tilt sensor, OSG (catch) and the plastic wheels. Inspection must be performed by an authorised service technician.

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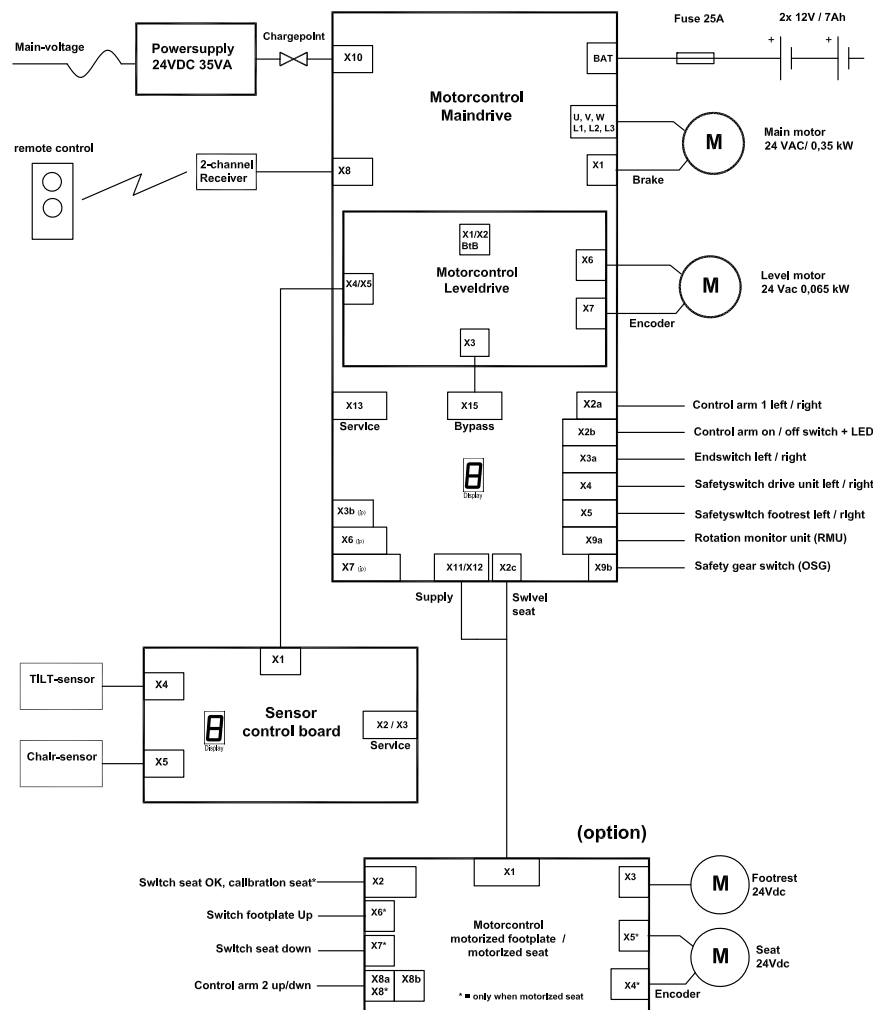
EN

Technical specifications General

Electrical system
Ambient conditions

Maximum weight capacity 125 kg
Battery power time Up to 8 hours
Fire resistance EN 81-40
Power voltage 24 Volt, 1.4 AMP.
Mains 230V, 0.15 AMP
Ambient temperature 0 to 40°C
Relative humidity 90% RH or less (non-condensing)

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EN

Otolift Modul-Air environment and batteries

The lift contains two batteries, each 12 V (7 Ah), to provide the lift with power. These batteries may only be replaced by authorized maintenance technicians. The remote controls are equipped with a penlight AA battery. You may replace these batteries yourself (see page 50). Do not dispose of old batteries with domestic refuse, but hand them in when you buy the new batteries.

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EN

Otolift Modul-Air and removal of the lift

When an Otolift is no longer required the supplying dealer may take it back. However this is purely at their discretion. They are under no obligation to do so. Contact them in the first instance for advice. If this is not possible, find out from your local government administration what the possibilities are for environmentally friendly disposal of the materials.

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